



EPA Reg. No. 8383-7-56753

Surface Disinfectant Wipes

CiDecon® Plus Wipes are disinfectant wipes in a convenient pop-up canister. An effective disinfectant and active deodorant for laboratories, hospitals, clinics, medical, dental, and veterinary offices, industrial clean rooms, homes, nursing homes, ambulances, etc. For use on hard non-porous, inanimate surfaces, provides broad range efficacy against bacteria, virus, and fungi. Proven effective against tuberculosis, HIV-1, and many other pathogenic organisms. CiDecon® Plus Wipes are safe, effective and meet OSHA bloodborne pathogen standard. The wipe is made with non-woven cellulose polyester (80% Poly/20% Rayon).



180 Wipes per canister
12 canisters per case
Decon Catalog No.
8511

ACTIVE INGREDIENTS

Phenol	1.56%
Sodium Phenate	0.06%
Inert ingredients	98.38%
Total	100.00%

DIRECTIONS FOR DISPENSING: Pull tab and remove cover from container and pull up the wipe corner from the center. Twist the corner into a point and insert about one inch of the wipe through the X hole on the underside of the cover. Pull the wipe completely through and replace cover onto container. Pull out first wipe at a 90° angle. Remaining wipes feed automatically. Always reseal the hinged cap after each use to ensure moisture retention.

USE PROTOCOL: PRECLEAN surface to remove soil and filth. Thoroughly wet pre-cleaned surface and allow surface to remain wet for 10 minutes at room temperature to kill listed organisms.

FOR HOSPITAL DISINFECTION:

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

4 YR SHELF LIFE

Efficacy Claims:

PASSES A.O.A.C. EFFICACY STANDARDS FOR INSTITUTIONAL AND HOSPITAL DISINFECTION.

VIRUCIDAL:

- Avian Influenza A (H9N2)
- Influenza A2 virus (Japan 305/57 Asian strain)
- Cytomegalovirus
- Coronavirus
- Polio type 1 virus
- Vaccinia virus
- Herpes Simplex types 1/F and 2/G (oral, ocular and genital)
- HIV-1 (Aids Virus)

BACTERICIDAL:

- Staphylococcus aureus
- Salmonella enterica
- Pseudomonas aeruginosa
- Methicillin resistant Staphylococcus aureus
- Vancomycin resistant Enterococcus faecium (VRE)
- E. coli
- Streptococcus pyogenes
- Streptococcus salivarius
- Mycobacterium tuberculosis

FUNGICIDAL:

Trichophyton mentagrophytes (Athlete's Foot Fungus)

STORAGE AND DISPOSAL

Store at room temperature. Non- refillable container. Do not use or refill this container. Discard empty container in trash or offer for recycling if available.

PRECAUTIONARY STATEMENTS

HAZARDOUS TO HUMANS AND DOMESTIC ANIMALS

KEEP OUT OF REACH OF CHILDREN

CAUTION: Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling.

FIRST AID

IF IN EYES: Remove contact lenses if present. Immediately flush eyes with plenty of water for at least 15 to 20 minutes. Ensure adequate flushing of the eyes by separating the eyelids with fingers. Get immediate medical attention.

IF ON SKIN OR CLOTHING: Immediately wash skin with plenty of soap and water for 15 to 20 minutes, while removing contaminated clothing and shoes. Get medical attention if irritation develops or persists.

IF INHALED: If inhaled, remove to fresh air. If not breathing, give artificial respiration or give oxygen by trained personnel. Seek immediate medical attention.

IF SWALLOWED: If swallowed, do NOT induce vomiting. Call a physician or poison control center immediately. Never give anything by mouth to an unconscious person.

HOT LINE NUMBER: In case of an emergency call toll free Chemtrec at 800-424-9300. Have the product container or label with you when calling a poison control center or doctor or going for treatment.

NOTE TO PHYSICIAN: Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

