



SlideRite™

WATER SOLUBLE FIXATIVE SPRAY Technical Data Sheet

– BRIEF DESCRIPTION –

Slide Rite is a highly concentrated, water-soluble spray that forms a lasting protective coating over all types of cytological smears. Many cytological specimens are fragile and degenerate quickly. Therefore an immediate fixation of the material to be studied is essential.

– USE PROTOCOL –

After spreading the specimen evenly and smoothly on the glass slide, immediately spray the Slide Rite. Hold the can six inches from the microscope slide and spray over the specimen slowly and evenly using a continuous flow of product until the entire specimen is coated. The protective coating will dry in approximately six minutes. When dry, the specimen is protected by a clear film and is ready for transport or storage.

– INGREDIENTS –

Isopropyl Alcohol 84.2 %
Polyethylene Glycol6.4 %

– STORAGE AND DISPOSAL –

Storage: Store container between 10⁰ C. and 30⁰ C. (50⁰ F and 85⁰ F). Do not expose can to extreme heat or temperatures above 120⁰ F.

Disposal: Do not puncture or incinerate container. Dispose in accordance to federal, state, and local regulations.

– PACKAGING –

1.5 oz./net wt. 42.5 grams 12/case

– CATALOG NUMBERS –

| <u>Manufactured for:</u> | <u>Item</u> | <u>Cat. No.</u> |
|--------------------------|-------------|-----------------|
| Decon Labs, Inc. | 1.5 oz. | 1801 |

For Additional Technical Information call
Decon Customer Service
(800) 332-6647, or
See our website at www.deconlabs.com

™ Decon Labs, Inc. Revised 11/2006